IMPORTANT NOTICE  
AVIS IMPORTANT  
WICHTIGER HINWEIS  
AVISO IMPORTANTE  
BELANGRIJKE INFORMATIE  
COMUNICAZIONE IMPORTANTE  
VIKTIG MERKNAD  
TÄRKEÄ ILMOITUS  
VIKTIG INFORMATION  
VIGTIG INFORMATION  
AVISO IMPORTANTE  
POMEMBNO OBVESTILO  
DŮLEŽITÉ UPOZORNĚNÍ  
FONTOS INFORMÁCIÓK  
WAŻNA INFORMACJA  
ÖNEMLİ DUYURU  
DÔLEŽITÉ UPOZORNENIE  
ΣΗΜΑΝΤΙΚΗ ΓΝΩΣΤΟΠΟΙΗΣΗ  
ВАЖНОЕ ЗАМЕЧАНИЕ  
AVISO IMPORTANTE  
VAŽNA OBAVIJEST  
VAŽNA NAPOMENA  
ВАЖНА ИНФОРМАЦИЯ  
AVIZ IMPORTANT  
OLULINE MÄRKUS  
MIKILVÆG TILKYNNING  
SVARBI PASTABA  
SVARĪGS PAZIŅOJUMS  
NJOFTIM I RĒNDĒSISHĒM  
ВАЖНО ИЗВЕСТУВАЊЕ  
ВАЖЛИВЕ ЗАУВАЖЕННЯ  
重要说明  
중요 고지  
留意事項  
NOTIS PENTING  
PEMBERITAHUAN PENTING

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## ENGLISH

### USING THE SET CHANGE REMINDER WITH THE MEDTRONIC EXTENDED INFUSION SET AND EXTENDED RESERVOIR

**The Extended infusion set (all model types for MMT-431A, MMT-432A, MMT-441A, MMT-442A, MMT-443A) and Extended reservoir (all model types for MMT-342) can be used for up to seven days with the insulin pump models listed here.**

The Set Change reminder alerts you when it is time to change your infusion set and reservoir. The Extended infusion set and Extended reservoir can be used with your pump for up to a maximum of seven days. The Set Change reminder can only be programmed for up to three days and should be turned off to avoid confusion. The Set Change reminder in the pump is set to off by default. If you have turned on this setting, follow these steps to turn it off:

#### **MiniMed™ 780G Pump**

1. Press  and then select .  
**Alert Settings > Reminders > Set Change**
2. Select **Reminder** to turn the reminder off.
3. Select **Save**.

#### **MiniMed™ 700, 720G, and 740G Pump**

1. Press  and go to the Set Change screen.  
**Reminders > Set Change**
2. Select **Reminder** to turn the reminder off.
3. Select **Save**.

#### **MiniMed™ 770G, and 670G Pump**

1. Press  and go to the Set Change screen.  
**Options > Reminders > Set Change**
2. Select **Reminder** to turn the reminder off.
3. Select **Save**.

## MiniMed™ 640G Pump

1. Press  and go to the Set Change screen.

### **Reminders > Set Change**

2. Select **Reminder** to turn the reminder off.

3. Select **Save**.

**Note:** For warnings related to the Extended infusion set and the Extended reservoir, refer to the user guide provided with the devices.

Use only Medtronic infusion sets and reservoirs with your Medtronic insulin pump.

**Contact a local Medtronic support representative for assistance.**

© 2020 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.

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© 2020 Medtronic. All rights reserved. Medtronic, Medtronic 로고 및 Further, Together는 Medtronic의 상표입니다. 기타 모든 브랜드는 Medtronic 기업의 상표입니다.
© 2020 Medtronic. All rights reserved. Medtronic、Medtronicロゴおよび Further, Togetherは、Medtronicの商標です。その他の製品名はすべて、メドトロニック社の商標です。
Medtronic B.V.