


# WALLET CARD

MiniMed™ 780G System



## Main Menu

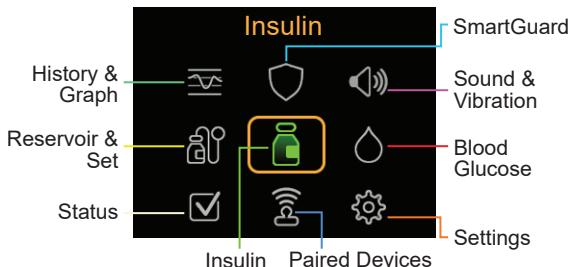

**History & Graph**

- History
- Sensor Glucose Review\*
- Graph
- Time in Range\*

**SmartGuard**

- SmartGuard Checklist
- Temp Target\*
- SmartGuard Settings\*
- SmartGuard\*

**Sound & Vibration**

- Silence Sensor Alerts\*
- Volume
- Sound
- Vibration
- Alert Settings

**Reservoir & Set**

- New Reservoir
- Fill Cannula

**Insulin**

- Bolus
- Basal
- Suspend All Delivery
- Delivery Settings

**Blood Glucose**

- Enter BG

**Status**

- Suspend All Delivery
- SmartGuard Checklist
- Pump

**Paired Devices**

- Pair New Device
- Pair CareLink

**Settings**

- Alert Settings
- Delivery Settings
- Device Settings

*\*In some cases, menu choices appear, but are greyed out if certain settings are turned on or off.*

## Home screen without CGM in Manual mode

status icons

09:00 — current time

6.1 — BG reading

mmol/L

Active Insulin 1.0 U — active insulin

## Home screen with CGM in Manual mode

status icons

09:00 — current time

9.2 — SG reading

mmol/L

0.5 U — active insulin


Act. Insulin

sensor graph

suspend by sensor icon

trend arrows

## Home screen in SmartGuard™ feature

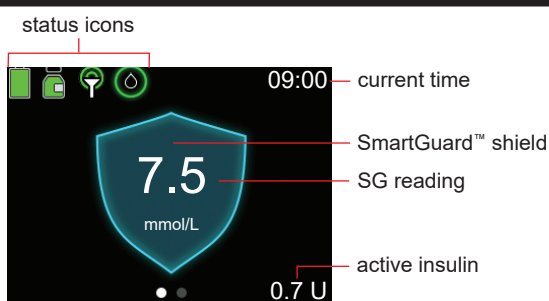



## Stopping and resuming insulin delivery

### Stopping all insulin delivery

The Suspend All Delivery option stops all insulin delivery (basal and bolus).



### To stop all insulin delivery:

1. From the Home screen, press  and select .
2. Select **Suspend All Delivery**.
3. Select **Yes** when prompted.

### Resuming basal insulin delivery

The Resume Basal option restarts the basal delivery after the pump has been suspended. The Resume Basal option does not restart a bolus delivery.

### To resume basal insulin delivery:



1. From the Home screen, press  and select .
2. Select **Resume Basal**.
3. Select **Yes** when prompted.

**NOTE:** Basal insulin delivery can be manually resumed during a Suspend before low or Suspend on low event.



### Stopping a bolus delivery

The Stop Bolus option stops a bolus delivery only. The Stop Bolus option does not stop the basal delivery.

### To stop a normal bolus, or the Now portion of a Dual Wave™ bolus:

1. From the Home screen, press  and select .
2. Select **Stop Bolus**.
3. Select **Yes** when prompted.
4. Select **Done**.



### To stop a Square Wave™ bolus or the Square portion of a Dual Wave™ bolus:

1. From the Home screen, press  and select .
2. Select **Bolus**.
3. Select **Stop Bolus**.
4. Select **Yes** when prompted.
5. Select **Done**.

### Manually resuming basal insulin delivery during a Suspend before low or Suspend on Low event.

If unable to wait for the pump to automatically resume the basal insulin during a Suspend before low or Suspend on low event, manually resume the basal delivery.

### To manually resume basal insulin delivery:

1. From the Home screen, press  and select .
2. Select **Resume Basal**.
3. Select **Resume Basal** again.
4. Select **Yes** when prompted.

© 2020 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic.™™ Third party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company.

MiniMed™, SmartGuard™, Dual Wave™, and Square Wave™ are trademarks of Medtronic MiniMed, Inc.

**Medtronic**



**Medtronic MiniMed**  
2 Alma Road Macquarie Park,  
NSW, 2113 Australia  
1800 777 808

C €0459

M998613A021\_A