

Setting Temp Basal Rate

Device: MiniMed™ 780G insulin pump


You can set a temporary basal rate (TBR) in MiniMed™ 780G in Manual Mode. The temp basal feature helps set and start a temporary basal rate that can be used immediately to manage BG for short-term activities or conditions. This feature is independent of Continuous Glucose Monitoring, so you don't need to have a working sensor to use the TBR. The TBR is a way to temporarily raise or lower your basal rates and can be set for a range from 30 minutes to 24 hours. After the temp basal rate delivery is completed or cancelled, the programmed basal pattern resumes. The temp basal rates and preset temp basal rates can be defined using either a percentage of the current basal pattern or by setting a specific rate. Check with your healthcare professional regarding the use of Temp Target.

To set a Temp Basal Rate:

- From the Home screen, press , and then select .



- Select **Basal** and then **Temp Basal**

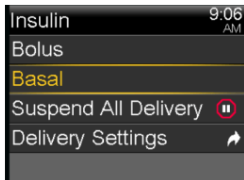


- Set the **Duration**



- Select **Next**

- Select **Type** to select choose **Rate** or **Percent**.

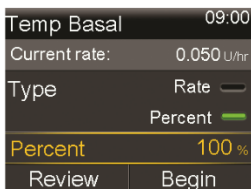

- Depending on the type selected, do one of the following:

- Enter a percentage (as seen above)

- Enter a basal rate

Select **Review** to review the temp basal settings

- Select **Begin** to start the temp basal rate.

The Temp Basal banner appears on the Home screen during delivery.



When the **Temp Basal** is complete, the basal will automatically return to the regularly programmed basal rate.

Always follow the directions for use

Information contained herein is not medical advice and should not be used as an alternative to speaking with your doctor. Discuss indications, contraindications, warnings, precautions, potential adverse events, and any further information with your healthcare professional.

©Medtronic. Medtronic, Medtronic logo, and Engineering the extraordinary are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.

2 Alma Road

Macquarie Park

NSW 2113

Tel: +61 2 9857 9000

Fax: +61 2 9889 5167 Toll Free: 1800

777 808 medtronic-diabetes.com.au