


Silencing glucose alerts


The Alert silence feature silences certain sensor alerts for a set period of time. When using this option, the Alert silence icon  appears on the Home screen. The system still displays any alerts that occur, but there is no sound or vibration if they are silenced. This information can be reviewed in the Alarm History screen.

The Alert silence feature does not silence:


- High SG alert - when your sensor glucose (SG) value is above 13.9mmol/L for more than three hours.
- Low SG alarm - when your SG value falls below 3.0mmol/L with Guardian™ 4 sensor.
- SmartGuard exit alert - when the pump exits the SmartGuard feature.

To Silence Sensor Alerts:

1. From the Home screen, press **Select**.
2. Select **Sound & Vibration**.

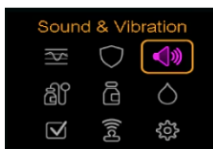

3. Select **Silence Sensor Alerts**.

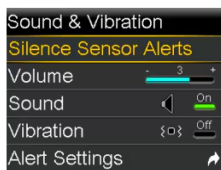

4. Select **High Alerts Only**, **High & Low Alerts** or, **All Sensor Alerts**.

Note: silencing all sensor alerts prevents the sound and vibration of most alerts related to SG readings, the sensor, and the transmitter. The Low SG alarm, for when SG has fallen below 3.0mmol/L with Guardian™ 4 sensor, the SmartGuard exit alert, and the High SG alert cannot be silenced.

5. Set the **Duration**. The duration can be set in 15-minute increments from 30 minutes to 24 hours.




6. Select **Begin**.




To cancel silence sensor alerts:

1. From the **Home screen**, press **Select**.
2. Select **Sound & Vibration**.

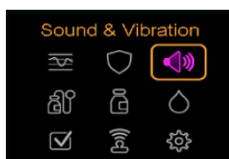

3. Select **Alert Silence**

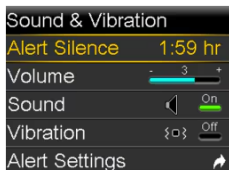

4. Select **Cancel Alert Silence**.



Always follow the directions for use

Information contained herein is not medical advice and should not be used as an alternative to speaking with your doctor. Discuss indications, contraindications, warnings, precautions, potential adverse events, and any further information with your healthcare professional.

©Medtronic. Medtronic, Medtronic logo, and Engineering the extraordinary are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.

2 Alma Road

Macquarie Park

NSW 2113

Tel: +61 2 9857 9000

Fax: +61 2 9889 5167

Toll Free: 1800 777 808

medtronic-diabetes.com.au