
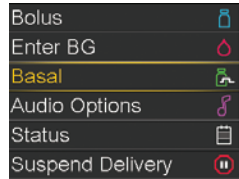
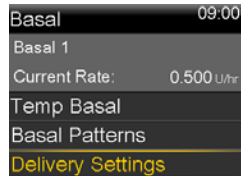


Change a basal rate


- From the Home screen, press .
- Select **Basal**.




- Select **Delivery Settings**.





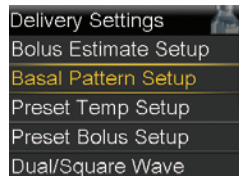


- Select **Basal Pattern Setup**.

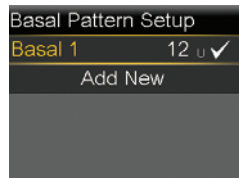


- Select the Basal Pattern you wish to edit.




- Select **Options**.
- Select **Edit**.


- Press  on the time segment.
- Press  on **End** time.
- Press  or  to change **U/hr** and press .

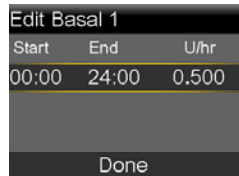


- Select **Done**.

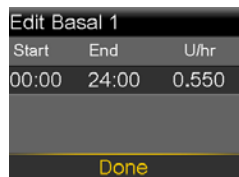


- Review rates and select **Save**.

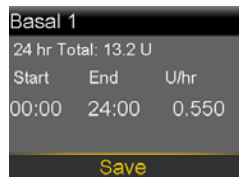


Reviewing Basal Patterns

- From the Home screen, press .
- Select **Basal**.

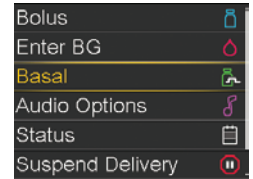


- Select **Basal Patterns**.

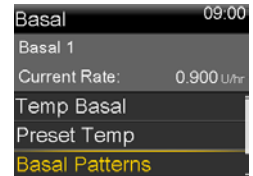

- Select the Basal Pattern you wish to review.




- Review basal rates.


NOTE: If you see a scroll bar on the right, press  to see all basal rates in the Basal Pattern.




- Select **OK**.

Add a basal rate to a Basal Pattern


- From the Home screen, press .
- Select **Basal**.

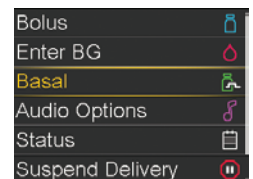

- Select **Delivery Settings**.



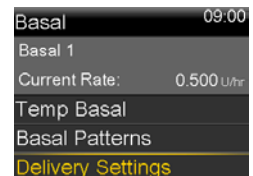


- Select **Basal Pattern Setup**.

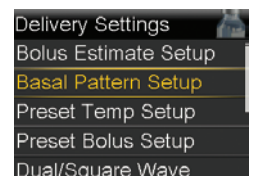

NOTE: The basal rates shown are for illustration purposes only—your basal settings will be different.

- Select the Basal Pattern you are adding a rate to.
- Select **Options**.
- Select **Edit**.
- Press on the time segment.
- Enter the new **End** time (this is the same as the start time of the basal rate you are adding) and press .
- Press if **U/hr** is not changing (Press or to change value and press .
- Press on the new time segment.
- Press to enter the new **End** time and press .
- Press to enter the new rate and press .
- Continue adding end times and basal rates if necessary.
- Select **Done**.
- Review basal rates.
- Select **Save**.

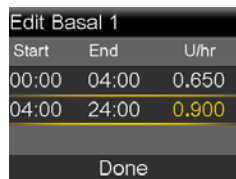
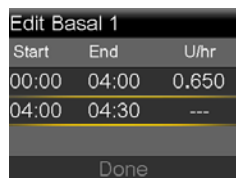
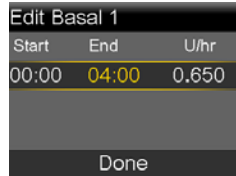


Temporary (Temp) basal rate

This feature lets you immediately increase or decrease your basal insulin for the period of time (duration) that you set. It is often used for exercise and sick days. A Temp Basal can be set in either Percent (delivers a percent of the current basal rate) or by Rate (delivers the amount that you enter).

Setting a Temp Basal


- From the Home screen, press .
- Select **Basal**.
- Select **Temp Basal**.
- Press to set duration and press .
- Select **Next**.
- Select **Percent**.
- Press or to enter the percent of current basal rate desired and press .
- Select **Begin**.

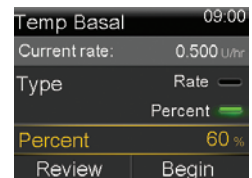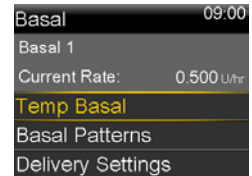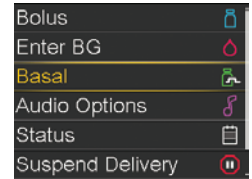

NOTE: If you choose to use Rate, press to Type and press .

The Home screen displays a **Temp Basal** banner to indicate that you have a Temp Basal active.


From the menu select **Cancel Temp Basal** to review the details of the active Temp Basal.

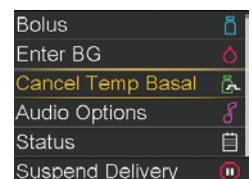


When the Temp Basal is complete, the basal delivery automatically returns to your regularly programmed basal rate.


Cancel Temp Basal rate

If you need to return to your regularly programmed basal rate before your Temp Basal is completed, you can cancel it.


1. From the Home screen, press .
2. Select **Cancel Temp Basal**.

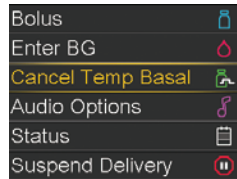

3. You can see the details about the Temp Basal.

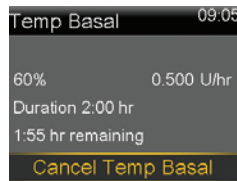

Select **Cancel Temp Basal**.


If you decide not to cancel, just press .


You can see that the Home screen no longer displays the Temp Basal Banner.

Multiple Basal Patterns


Setting multiple Basal Patterns helps you more easily accommodate routine schedule changes that cause different basal needs (for example, weekday vs. weekend; day vs. night shift).

Set an additional Basal Pattern

1. From the Home screen, press .
2. Select **Basal**.

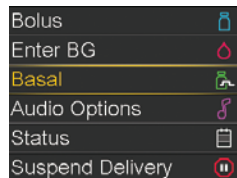


3. Select **Delivery Settings**.
4. Select **Basal Pattern Setup**.

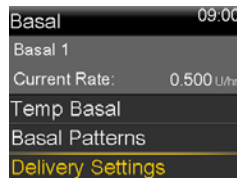


5. Select **Add New**.




6. Select the name you would like to use.
7. Enter the basal rates needed for this pattern.
8. Select **Save**.

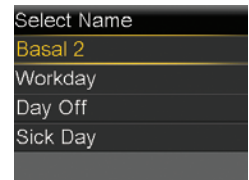


NOTE: The Basal pattern that your pump is currently using has a checkmark next to it.

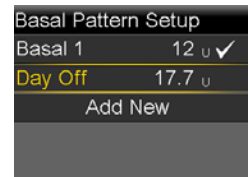


How to change which Basal Pattern is Active


1. From Home screen, press .
2. Select **Basal**.

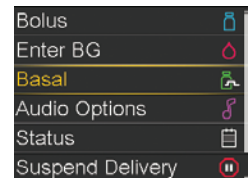

3. Select **Basal Patterns**.

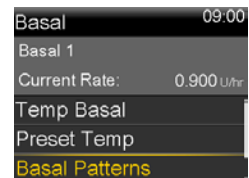


4. Select the Basal Pattern you wish to be active.

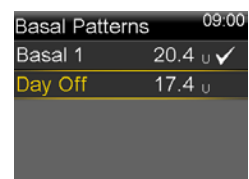

NOTE: The checkmark indicates which Basal Pattern is active.

5. Select **Begin**.

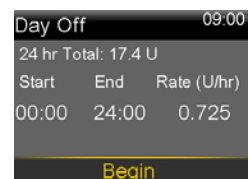

Always read the instructions for use

NOTE: The basal rates shown are for illustration purposes only—your basal settings will be different.